



ARREDAMENTO PER L'ESTETICA 100% MADE IN ITALY



## **PODOLUX**

**MB/P30**

**INSTRUCTION MANUAL**

### Safety during installation

- Before installing and using the treatment chair, carefully read the instruction manual and ensure that all directions and instructions contained are fully understood. It includes important safety information for installation, operation and maintenance!
- When unpacked, check the contents of the delivery to ensure that all parts have been included and that nothing has been damaged during transit. In the event of any doubt, please contact our customer service department.
- The Podolux treatment chair must be positioned on a flat surface, levelled and able to bear the weight when the chair is loaded, referring to the maximum allowable load.
- Before connecting to the mains power supply ensure that the voltage (V) shown on the type plate corresponds with the power supply available!

### Safety during use

- If the treatment chair is used for any purpose other than that for which it was designed and as described in this instruction manual, such use will invalidate the manufacturers warranty and guarantee!
- This unit may only be used by personnel that complete the treatment from hereon 'operator'.
- Children should not be left unattended in proximity to the equipment.
- Do not, for any reason, place feet, hands or other body parts under the moving part of the frame even when motionless.
- Under no circumstances may the mains plug be disconnected from the socket with wet hands. Never attempt to remove the plug from the socket by pulling the connecting cable!
- In the event of any liquid or foreign body entering the inner of the control unit, immediately disconnect the treatment chair from the mains power supply! Please consult our customer service department before using again.
- To disconnect the equipment, all movements should be stopped. Only then can the plug be removed from the electrical socket.
- Pay the utmost attention when the shampoo unit is used for children, senior or disabled people.

### Safety during maintenance

- Before any maintenance or cleaning operation, unplug the unit.
- Cleaning and maintenance must be made only by those qualified for such a job.
- Keep children away from the moving part of the frame.
- If the moving parts (frame and footrest) of the shampoo unit jam, do not, for any reason, try to clear them, but rather contact Medical & Beauty (Salon Habitat Italia srl) customer support.

### Prohibited use

Never use the Podolux treatment chair:

- If the power supply cable and / or the plug are damaged.
  - If the case is visibly damaged.
  - After the unit has been overturned or fallen on the ground.
- In all these cases, immediately contact Medical & Beauty (Salon Habitat Italia srl) customer support for the necessary repairs.
- Do not let objects or water fall into the openings and/or the cracks.
  - Always keep the power cord away from heat or hot surfaces.
  - Do not use the treatment chair outdoors, in damp surroundings, near environments where spray (aerosol) is used, or near environments where oxygen is stored and used.
  - To unplug, stop all movements and then remove the plug from the socket.

### Fields of use

The Podolux treatment chair is a very comfortable treatment chair for cosmetic treatments. The chair is equipped with electric adjustment of the height of the seat, backrest and tilting seat. Gas pistons regulate the legrests. Together with the possibility of adjusting the seat height, the fully overturnable armrests make it extremely easy to get on and off the chair. The armrests have been designed to be overturned so as to offer maximum comfort in all positions.


### Components

Fig.1 shows the main components of the Podolux beauty treatment chair.

Legend for treatment chair

1. Headrest and backrest
2. Armrests
3. Legrests
4. Footpedal for height, seat tilting and backrest adjustment
5. Remote control for height, seat tilting and backrest adjustment

Fig. 1

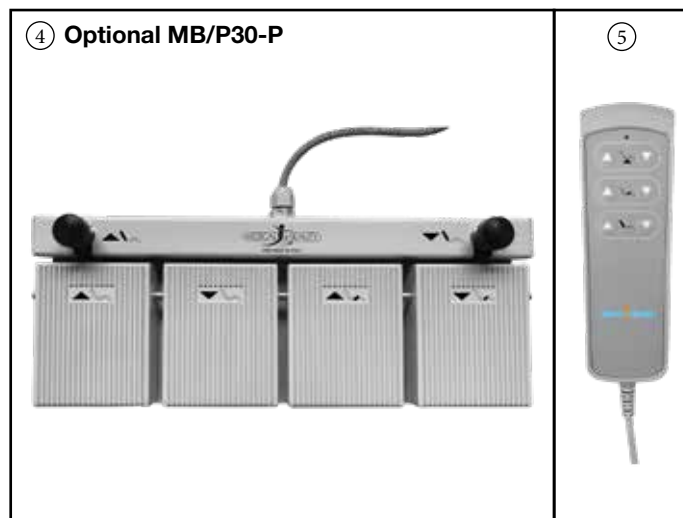

### Connecting the Podolux chair to the current

- Check that the mains voltage (V) corresponds to that shown on the data plate of the machine.
- Connect the plug to the socket.

### Operation of Podolux

**Caution! Risk of pinching! The extremities of the client must remain on the couch during adjustment of the reclining position.**

Do not allow that oil or any care products come in contact with the upholstery because this can damage it. We recommend the use of an upholstery protection, e.g. a towelling cover. If oil comes in contact with the upholstery, remove immediately.

Adjustment operations can be controlled either by push-button panel or by foot pedal.


### Seat height adjustment:

Use the controls shown below to raise and lower the seat position as needed.



### Tilting seat adjustment:

Use the controls shown below to move the seat forwards and backwards as needed.

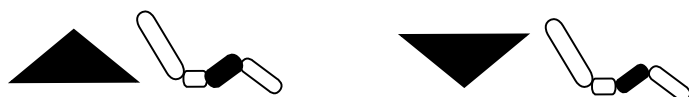

### Backrest adjustment.

Use the controls shown below to adjust the position of the backrest until the desired position is reached.



### Legrest adjustment:

The legrest can be adjusted independently from the backrest and seat, and can be adjusted:

- in length with an extension knob
- in height with lifting lever
- In opening

### Armrests adjustment

The position of the armrests is synchronized with the movement of the backrest, and can be raised or overturned to make it easy to get on and off the chair.

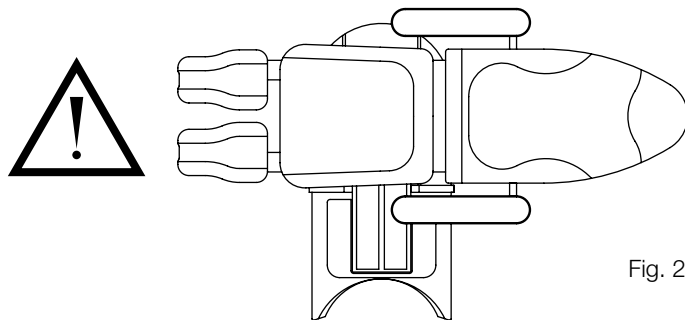


Fig. 2

**Caution! Never use the reclining chair as illustrated in fig. 2 DANGER OF ROLLOVER**

### Cleaning and maintenance

**Caution! Danger of electric shock! Disconnect from the mains power supply before cleaning.**

Clean the upholstery of the treatment chair with warm water and a cloth.

Remove larger areas of soiling with a mild cleaning agent.

Stains should be removed immediately to avoid them penetrating the material.

Clean the frame of the treatment chair and the manual control or the foot switch with a lightly moistened cloth.

Ensure that no water enters the control unit, the manual control or foot switch.

Do not rub the disinfectant in the upholstery.

Do not use any solvents, chlorides, polish, synthetic cleaning agents, wax polish, aerosol sprays or aggressive cleaning agents which could cause damage to the upholstery and the treatment chair.

Do not use any substances which contain oil or grease to clean the upholstery. The upholstery is not resistant to chemical cleaning and dry-cleaning methods.

### Repairs and Servicing

Your treatment chair does not require any special servicing when used correctly and in compliance with the manufacturer's directions and instructions. However, in the event of any problem arising, please contact our customer service department.

Only persons authorised by the manufacturer may perform repairs. Any alterations or repairs performed by non-authorised persons will invalidate any guarantee, warranty or responsibility of the manufacturer. In addition, the safety of the treatment chair could be adversely affected.

### Transport/ Storage/ Operating requirements

Transport only in the manufacturer's original packaging.

Transport and storage temperature: -20 °C to +50 °C

Operating temperature: +10 °C to +40 °C

Relative air humidity: max. 90 %, non-condensing

To avoid the formation of condensation, abrupt changes of temperature should be avoided. Wait until the treatment chair has acclimatised to the ambient temperature before putting into operation.

### Disposal

Dispose of the packaging, treatment chair, upholstery and accessories in an environmentally friendly manner and in compliance with local regulations. All electrical and electronic products should be disposed of separately from the municipal waste.



## DICHIARAZIONE DI CONFORMITA'

Noi: Salon Habitat (Italia) s.r.l.  
Via Lazzaretti 12  
42122 Reggio Emilia

Attestiamo sotto la nostra completa responsabilità che il prodotto:

**Poltrona elettrica per trattamenti Podolux MB/P30**

cui la dichiarazione si riferisce è conforme alle seguenti norme:

**EN 60335-1, modifiche 2, 5, 6, A51, A53, A55  
EN 60335-2-23, modifica A**

ed ai requisiti delle seguenti direttive CEE:

**Direttiva CEE 73/23 del Consiglio del 9 febbraio 1973; Attuazione della  
direttiva con D.L. del 18 ottobre 1977.  
Direttiva CEE 93/68 del Consiglio del 22 luglio 1993; modifica alla 73/23  
CEE**

Reggio Emilia, 01/01/2022

Il legale rappresentante  
Sig. Michael Frost

Salon Habitat (Italia) s.r.l. via Lazzaretti 12 42122 Reggio Emilia  
Sales Italia 0522 276740 International Sales 0522 276755 Fax 0522 276767  
info@medicalandbeauty.com • www.medicalandbeauty.com



**Manufacturer**

Salon Habitat (Italia) s.r.l. via Enzo Lazzaretti, 12  
42122 Reggio Emilia - Italy  
www.salonambience.com  
info@salonambience.com

**Model**

PODOLUX MB/P30 & MB/P31

**Function**

Beauty center massage chair/bed with  
electric mechanism.

**IEC Protection Class - II**

**Year of Manufacture - 2020 - 2021 - 2022**

**Power Supply**

100/220-240 V~ 50-60 Hz



**Nominal Power**

90W

**Maximum allowable load for the chair - 130 Kg**