



ARREDAMENTO PER L'ESTETICA 100% MADE IN ITALY



Dallas

MB/P10 - MB/P11

INSTRUCTION MANUAL

Safety during installation

- Before installing and using the treatment chair, carefully read the instruction manual and ensure that all directions and instructions contained are fully understood. It includes important safety information for installation, operation and maintenance!
- When unpacked, check the contents of the delivery to ensure that all parts have been included and that nothing has been damaged during transit. In the event of any doubt, please contact our customer service department.
- The Dallas treatment chair must be positioned on a flat surface, levelled and able to bear the weight when the chair is loaded, referring to the maximum allowable load.
- Before connecting to the mains power supply ensure that the voltage (V) shown on the type plate corresponds with the power supply available!

Safety during use

- If the treatment chair is used for any purpose other than that for which it was designed and as described in this instruction manual, such use will invalidate the manufacturers warranty and guarantee!
- This unit may only be used by personnel that complete the treatment from hereon 'operator'.
- Children should not be left unattended in proximity to the equipment.
- Do not, for any reason, place feet, hands or other body parts under the moving part of the frame even when motionless.
- Under no circumstances may the mains plug be disconnected from the socket with wet hands. Never attempt to remove the plug from the socket by pulling the connecting cable!
- In the event of any liquid or foreign body entering the inner of the control unit, immediately disconnect the treatment chair from the mains power supply! Please consult our customer service department before using again.
- To disconnect the equipment, all movements should be stopped. Only then can the plug be removed from the electrical socket.
- Pay the utmost attention when the shampoo unit is used for children, senior or disabled people.

Safety during maintenance

- Before any maintenance or cleaning operation, unplug the unit.
- Cleaning and maintenance must be made only by those qualified for such a job.
- Keep children away from the moving part of the frame.
- If the moving parts (frame and footrest) of the shampoo unit jam, do not, for any reason, try to clear them, but rather contact Medical & Beauty (Salon Habitat Italia srl) customer support.

Prohibited use

Never use the Dallas treatment chair:

- If the power supply cable and / or the plug are damaged.
- If the case is visibly damaged.
- After the unit has been overturned or fallen on the ground. In all these cases, immediately contact Medical & Beauty (Salon Habitat Italia srl) customer support for the necessary repairs.
- Do not let objects or water fall into the openings and/or the cracks.
- Always keep the power cord away from heat or hot surfaces.
- Do not use the treatment chair outdoors, in damp surroundings, near environments where spray (aerosol) is used, or near environments where oxygen is stored and used.
- To unplug, stop all movements and then remove the plug from the socket.

Fields of use

The Dallas treatment chair is a very comfortable treatment chair for cosmetics, massage and foot care. The Dallas can be moved into a variety of positions, with the chair height regulated by an electric foot pedal, the angle of the seat and backrest regulated by two separate gas pedals, and the two separate leg rests adjustable manually. The chair comes complete with two armrests that are rotatable, allowing them to be utilised when the Dallas is positioned as either as a chair or a bed. The leg rests and headrest can also be extended to allow greater comfort for customers of any type.


Components

Fig.1 shows the main components of the Dallas beauty treatment chair.

Legend:

1. Head rest
2. Back rest
3. Arm rests
4. Leg rest
5. Foot rest
6. Footpedal for height adjustment (up/down)
7. Footpedal for seat adjustment
8. Footpedal for backrest adjustment

Fig. 1

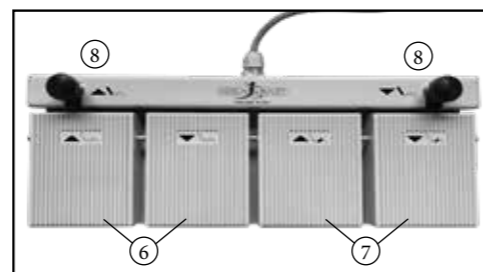

Connecting the Dallas chair to the current

- Check that the mains voltage (V) corresponds to that shown on the data plate of the machine.
- Connect the plug to the socket.

Operation of Dallas

Caution! Risk of pinching! The extremities of the client must remain on the couch during adjustment of the reclining position.

Do not allow that oil or any care products come in contact with the upholstery because this can damage it.

We recommend the use of an upholstery protection, e.g. a towelling cover.

If oil comes in contact with the upholstery, remove immediately.

Headrest adjustment

Pull the headrest out until the required position is reached.

Push the headrest down to reset it.

Seat height adjustment:

Use the controls shown below to raise and lower the seat position as needed.



Tilting seat adjustment:

Use the controls shown below to move the seat forwards and backwards as needed.



Backrest adjustment.

Use the controls shown below to adjust the position of the backrest until the desired position is reached.



Leg rest adjustment

The leg rest is adjusted independently from the back rest and seating angle.

Turn the fixing lever beneath the seat section anti-clockwise to release the leg rest.

Move the leg rest into the required position with your hand.

Rotate the fixing lever clockwise to arrest the leg rest.

Foot rest adjustment

Attention! The foot rest must always be fully withdrawn in the entry and exit position (lowest position).

Pull the foot rest out until the required position is reached.

Push the foot rest back again.

Arm rests adjustment

Release the fixing screw under the arm rest.

Swing the arm rest sideways until the required position is reached.

Arrest the fixing screw again.

Cleaning and Maintenance

Caution! Danger of electric shock! Disconnect from the mains power supply before cleaning.

Clean the upholstery of the treatment chair with warm water and a cloth.

Remove larger areas of soiling with a mild cleaning agent.

Stains should be removed immediately to avoid them penetrating the material.

Clean the frame of the treatment chair and the manual control or the foot switch with a lightly moistened cloth.

Ensure that no water enters the control unit, the manual control or foot switch.

Do not rub the disinfectant in the upholstery.

Do not use any solvents, chlorides, polish, synthetic cleaning agents, wax polish, aerosol sprays or aggressive cleaning agents which could cause damage to the upholstery and the treatment chair.

Do not use any substances which contain oil or grease to clean the upholstery. The upholstery is not resistant to chemical cleaning and dry-cleaning methods.

Repairs and Servicing

Your treatment chair does not require any special servicing when used correctly and in compliance with the manufacturer's directions and instructions. However, in the event of any problem arising, please contact our customer service department. Only persons authorised by the manufacturer may perform repairs. Any alterations or repairs performed by non-authorised persons will invalidate any guarantee, warranty or responsibility of the manufacturer. In addition, the safety of the treatment chair could be adversely affected.

Transport/ Storage/ Operating Requirements

Transport only in the manufacturer's original packaging.

Transport and storage temperature: -20 °C to +50 °C

Operating temperature: +10 °C to +40 °C

Relative air humidity: max. 90 %, non-condensing

To avoid the formation of condensation, abrupt changes of temperature should be avoided. Wait until the treatment chair has acclimatised to the ambient temperature before putting into operation.

Disposal

Dispose of the packaging, treatment chair, upholstery and accessories in an environmentally friendly manner and in compliance with local regulations.

All electrical and electronic products should be disposed of separately from the municipal waste.



DICHIARAZIONE DI CONFORMITA'

Noi: Salon Habitat (Italia) s.r.l.
Via Lazzaretti 12
42122 Reggio Emilia

Attestiamo sotto la nostra completa responsabilità che il prodotto:

Poltrona elettrica per trattamenti Dallas MB/P10, MB/P11

cui la dichiarazione si riferisce è conforme alle seguenti norme:

**EN 60335-1, modifiche 2, 5, 6, A51, A53, A55
EN 60335-2-23, modifica A**

ed ai requisiti delle seguenti direttive CEE:

**Direttiva CEE 73/23 del Consiglio del 9 febbraio 1973; Attuazione della
direttiva con D.L. del 18 ottobre 1977.
Direttiva CEE 93/68 del Consiglio del 22 luglio 1993; modifica alla 73/23
CEE**

Reggio Emilia, 12/09/2024

Il legale rappresentante
Sig. Michael Frost